



Ella Big4Two & Tub4Two 2 Seat Acrylic Walk-In Bathtub Specifications

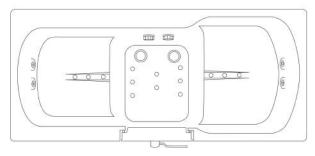

Big4Two - O2SA3680

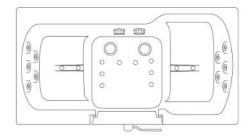

Tub4Two - O2SA3260-L, O2SA3260R

Actual Size: 36x80x42 Seat Width: 21/26 inches Shipping Size: 38x82x50 Shipping Weight: 350 lbs. Net Weight: 250 lbs.

Water Capacity: 145 gal Occupied: 85-135 gal Drainage Time: +/- 180 sec at ideal in home plumbing

conditions

Actual Size: 32x60x42 Seat Width: 19/22 inches Shipping Size: 34x62x50 Shipping Weight: 325 lbs.

Net Weight: 230 lbs.

Water Capacity: 100 gal Occupied: 65-85 gal

Drainage Time: +/- 120 sec at ideal in home plumbing

conditions

Included Features

- 1. Acrylic Tub Shell:
 - · Quality Acrylic Shell
 - Gloss White Finish
 - Fiberglass Reinforced
- 2. Stainless Steel Frame with Adjustable Leveling Legs
- 3. Plastic composite and tempered glass outward swing door with gear and shaft mechanics for 3 door latches
- 4. Two (2) front and one (1) end removable access panel

Floor Drains

 Two (2) 2" Brass threaded floor drains with copper overflow and cable operated opener with extended handle

Warranty

- Lifetime limited warranty on acrylic walk in tub shell, frame, and door
- 5 year parts warranty (please visit www.ellasbubbles.com for more detailed information)

Optional Equipment

- 1. Hydro Massage System 5AMP 600W
 - · Refer to technical drawing for jet count
 - Hydro massage intensity dial air control
 - · In-line water heater 5.5AMP 650W
 - · Ozone Sterilization
- 2. Air Massage System 4.5AMP 486W
 - · Refer to technical drawing for jet count
 - · 3 Speed Pump
 - Autopurge (Activates 20 minutes after pump shutdown)

Optional Equipment

- 3. MicroBubble Therapy System 10AMP 1200W
 - 1HP Pump
 - Pressure Vessel
 - · On/Off Push Control
 - · Agitator Valve
- 4. Heated Seat and Backrest 4AMP 470W
- 5. LED Chromatherapy Light .04AMP
- 6. Aromatherapy
- 7. On/Off Push Controls or Digital Controller
- 8. Five-Piece Thermostatic Control Valve Faucet Set*
 - 7-8 GPM (average) @ 60 PSI at ideal in home plumbing conditions
 - Two (2) 2-way diverters with extension handles
 - Pull out, hand held, chrome plated hand shower with 5 ft. retractable stainless steel braided hose
 - 1 year warranty

9. Five-Piece Fast Fill Huntington Brass Faucet Set


- 14-18 GPM (average) @ 60 PSI at ideal in home plumbing conditions
- Pull out, hand held, chrome plated hand shower with 5 ft. retractable stainless steel braided hose
- 5 year warranty visit www.huntingtonbrass.com/warranty for full warranty information


Electrical Requirements:

Please refer to Technical Diagrams for more information


*Available only on Tub4Two



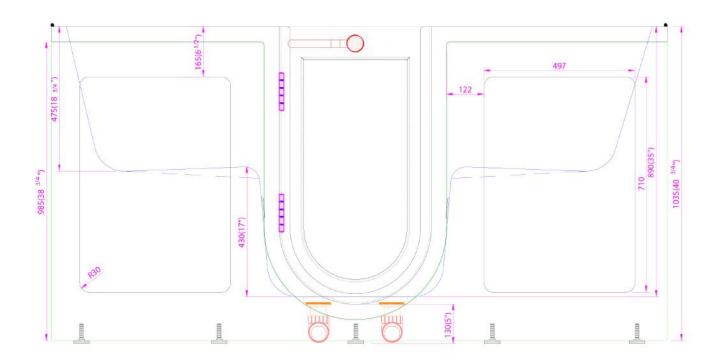


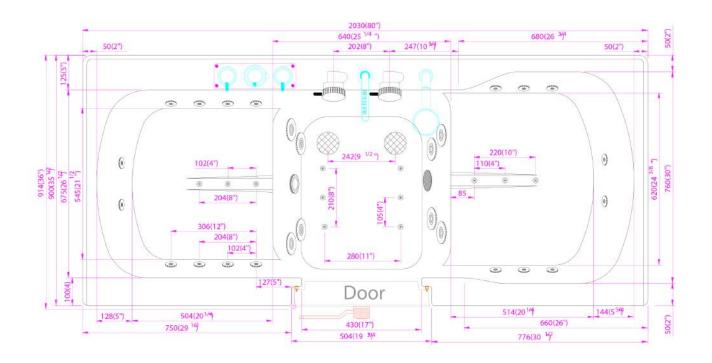




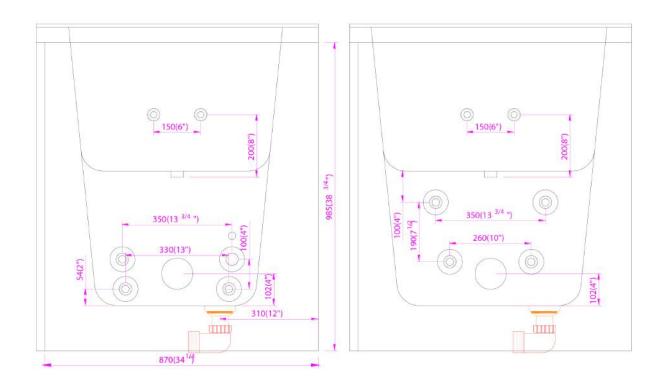




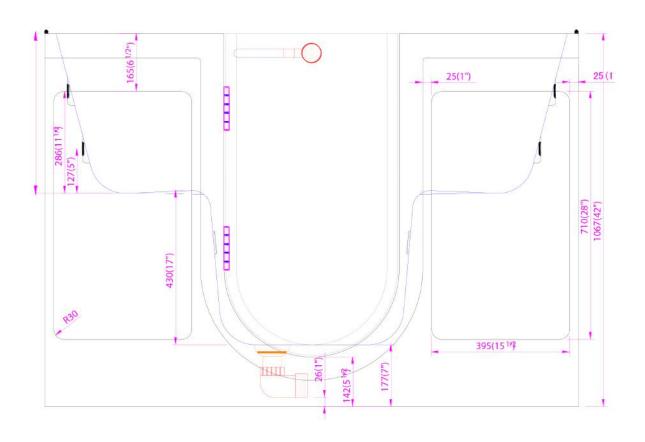

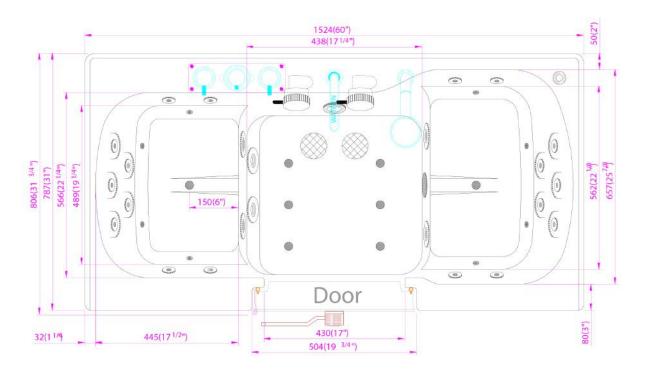




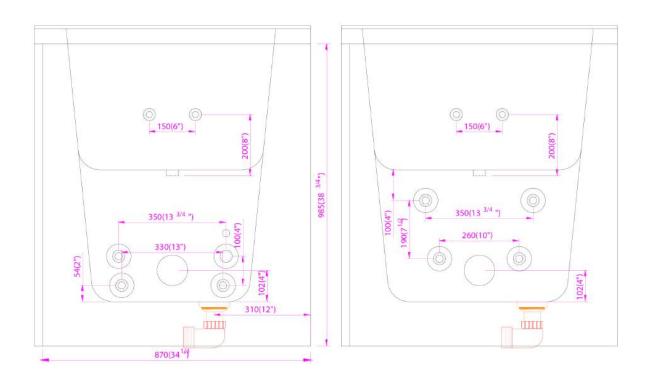



Hydro Massage + Air Massage (Dual)

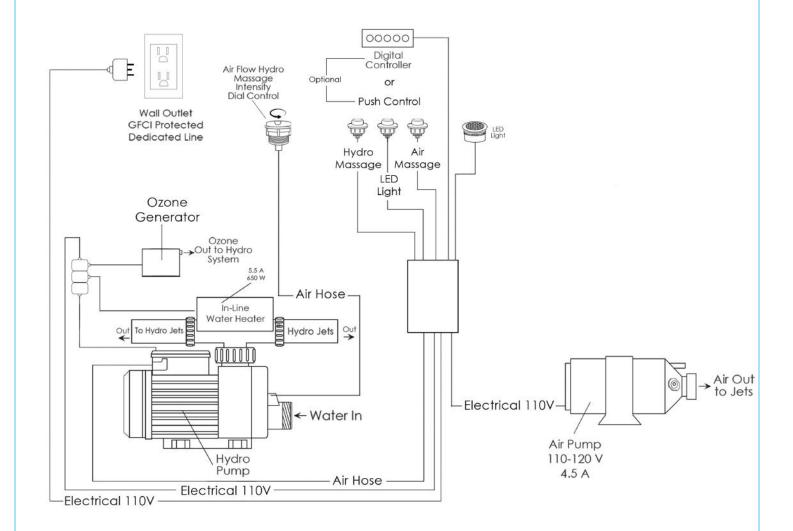

Package Jetting Information:

- 1. Hydro Massage System 5AMP 600W
 - Refer to technical drawing for jet count
 - Hydro Massage Air Flow Intensity Control
 - In-line water heater 5.5AMP 650W
 - Ozone Sterilization

2. Air Massage System 4.5AMP 486W


- · Refer to technical drawing for jet count
- · 3 Speed Pump
- Autopurge (Activates 20 minutes after pump shutdown)

Electrical Requirements:

^{*}For more information please visit www.ellasbubbles.com



Hydro Massage (Seat Area) + Air Massage + Hydro Massage (Foot Area)

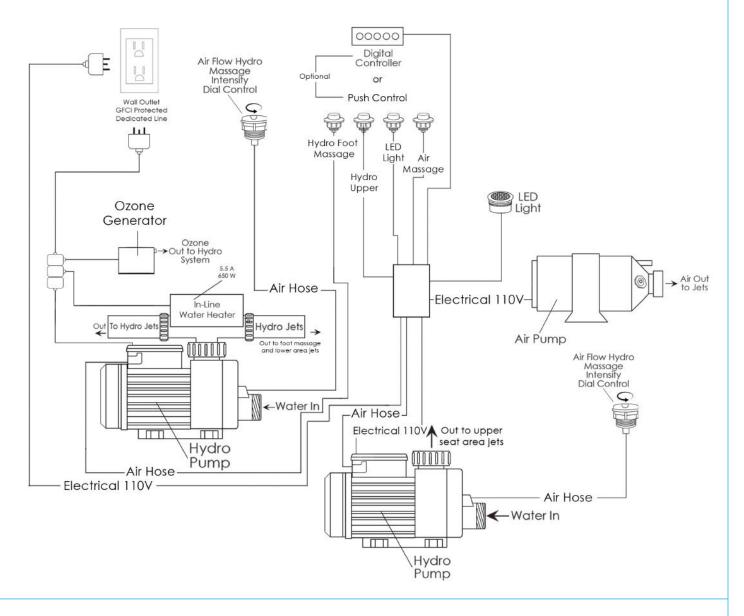

Package Jetting Information:

- 1. Hydro Foot Massage System 5AMP 600W
 - · Refer to technical drawing for jet count
 - Hydro Massage Air Flow Intensity Control
 - In-line water heater 5.5AMP 650W
 - · Ozone Sterilization
- 2. Air Massage System 4.5AMP 486W
 - Refer to technical drawing for jet count
 - · 3 Speed Pump
 - Autopurge (Activates 20 minutes after pump shutdown)

3. Hydro Massage System 5AMP 600W

- Refer to technical drawing for jet count
- Hydro Massage Air Flow Intensity Control


Electrical Requirements:

^{*}For more information please visit www.ellasbubbles.com



Hydro and Air Jet Massage with Microbubble Infusion Therapy Technical Diagram

Hydro Massage + Air Massage + Infusion Microbubble Therapy

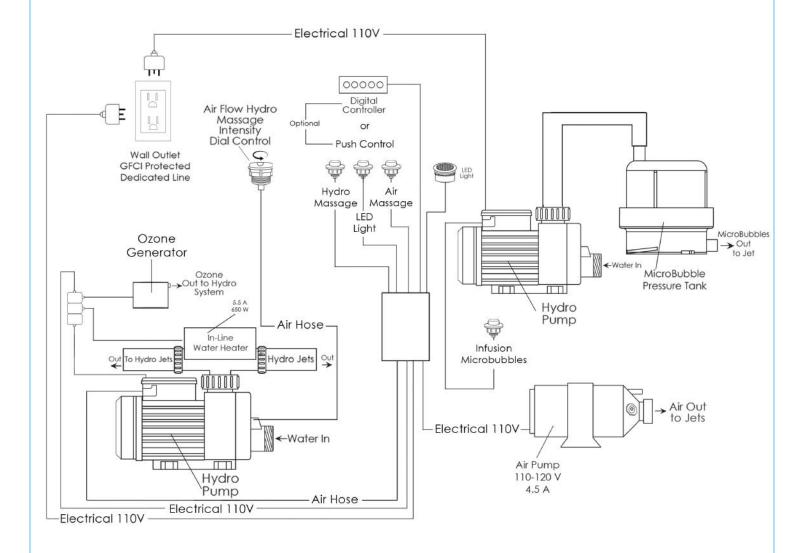

Package Jetting Information:

- 1. Hydro Massage System 5AMP 600W
 - Refer to technical drawing for jet count
 - Hydro Massage Air Flow Intensity Control
 - In-line water heater 5.5AMP 650W
 - Ozone Sterilization
- 2. Air Massage System 4.5AMP 486W
 - · Refer to technical drawing for jet count
 - · 3 Speed Pump
 - · Autopurge (Activates 20 minutes after pump shutdown)

3. MicroBubble Therapy System 10AMP 1200W


- 1HP Pump
- Pressure Vessel
- · Air Switch and one (1) jet
- Agitator Valve

Electrical Requirements:

^{*}For more information please visit www.ellasbubbles.com



Hydro Massage

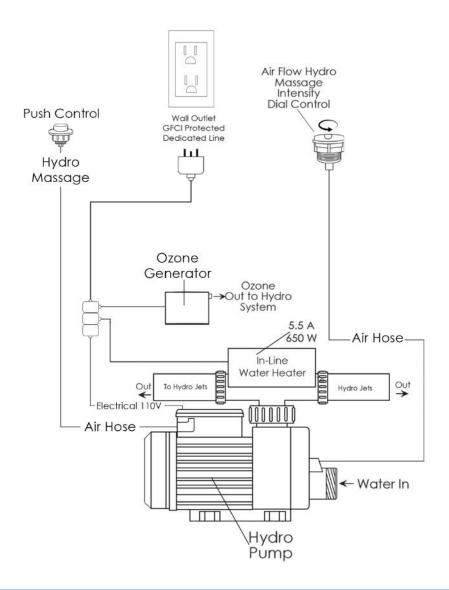

Package Jetting Information:


- 1. Hydro Foot Massage System 5AMP 600W
 - Refer to technical drawing for jet count
 - Hydro Massage Air Flow Intensity Control
 - In-line water heater 5.5AMP 650W
 - Ozone Sterilization

Electrical Requirements:



Hydro Jet Massage with Microbubble Infusion Therapy Technical Diagram

Hydro Massage + Infusion Microbubble Therapy

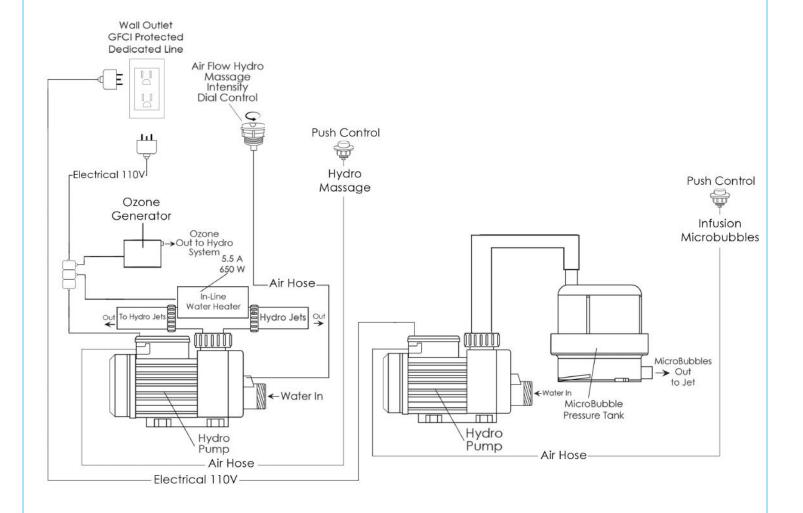

Package Jetting Information:

- 1. Hydro Massage System 5AMP 600W
 - Refer to technical drawing for jet count
 - Hydro Massage Air Flow Intensity Control
 - In-line water heater 5.5AMP 650W
 - Ozone Sterilization

2. MicroBubble Therapy System 10AMP 1200W

- 1HP Pump
- Pressure Vessel
- Air Switch and one (1) jet
- · Agitator Valve


Electrical Requirements:

One (1) dedicated 120V electrical line with a 20AMP GFCI standard three prong outlet is required

*For more information please visit www.ellasbubbles.com



Air Massage

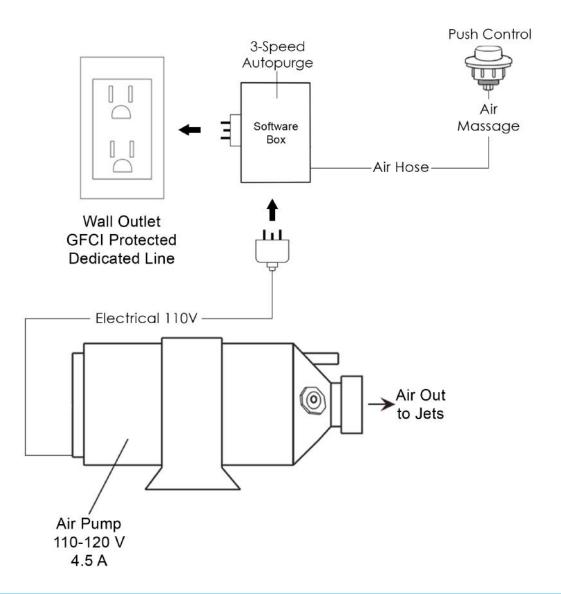

Package Jetting Information:


- 1. Air Massage System 4.5AMP 486W
 - Refer to technical drawing for jet count
 - · 3 Speed Pump
 - Autopurge (Activates 20 minutes after pump shutdown)

Electrical Requirements:



Air Jet Massage with Microbubble Infusion Therapy Technical Diagram

Air Massage + Infusion Microbubble Therapy

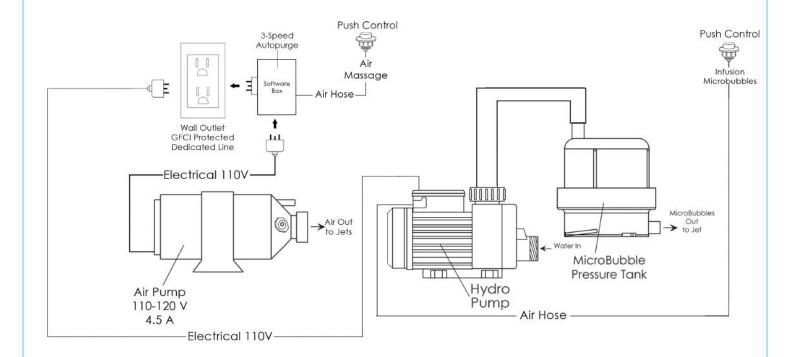

Package Jetting Information:

- 1. Air Massage System 4.5AMP 486W
 - · Refer to technical drawing for jet count
 - · 3 Speed Pump
 - Autopurge (Activates 20 minutes after pump shutdown)

2. MicroBubble Therapy System 10AMP 1200W

- 1HP Pump
- Pressure Vessel
- · Air Switch and one (1) jet
- Agitator Valve


Electrical Requirements:

One (1) dedicated 120V electrical line with a 20AMP GFCI standard three prong outlet is required

*For more information please visit www.ellasbubbles.com




Infusion Microbubble Therapy

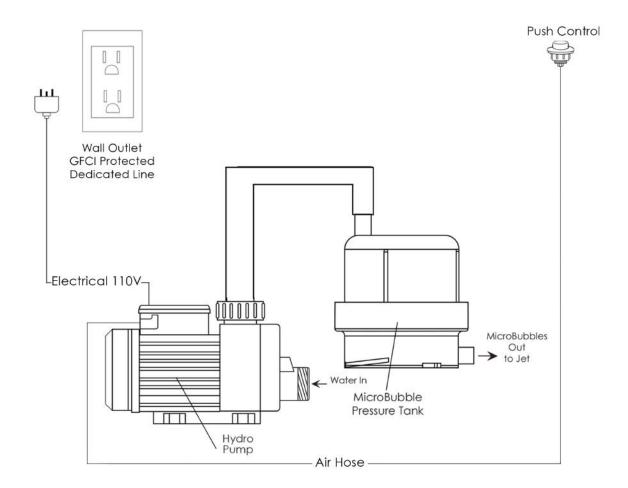

Package Jetting Information:

- 1. MicroBubble Therapy System 10AMP 1200W
 - 1HP Pump
 - Pressure Vessel
 - · Air Switch and one (1) jet
 - Agitator Valve

Electrical Requirements:

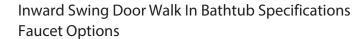



Thermostatic Control Valve 5 Piece Faucet Set



Color/Finish:

Chrome

Features:

- Adjustable water temperature preset
- No water temp fluctuation at the faucet
- Prevents scalding and thermal shock
- Standard on Walk In Tubs with an Ella 5 Piece Faucet Set
- 8 GPM (Average) @ 60 PSI at ideal in-home plumbing conditions
- 1 Year Warranty

Huntington Brass Fast Fill 5 Piece Faucet Set



Color/Finish:

Chrome

Features:

- Jewel Arched Solid Brass Spout
- 3/4" Valve, 3/4" Ports
- Ceramic Disc Cartidges
- Quick Connect Spout Installation
- NPS Connections
- 18 GPM (Average) @ 60 PSI at ideal in-home plumbing conditions
- 5 Year Warranty (visit www.huntingtonbrass.com/warranty for more information)





Seat Heater



On/Off Button



Backrest Heater





Electrical Requirements:

Voltage:

Input - 120V, 4AMP, 470W

SPECIFICATION / BREAK DOWN

5pc Roman Tub Filler HB9937P

Description

- 8" 16" Roman Tub Filler
- · Solid brass spout
- ceramic disc cartridges
- Quick connect spout installation
- NPS connections
- 18 GPM (average) @ 60 PSI

Standards

- ASME A112.18.1 Compliant
- Listed IAPMO / UPC & cUPC
- ADA Accessibility Guidelines for Buildings and Facilities - 4.27.4 Controls and Mechanisms







Colors / Finishes


C - Chrome NK- Satin Nickel

BREAK DOWN Component Parts List

| # | PART NAME | QTY |
|----|-----------------------------|-------|
| 1 | LEVER HANDLE | 3 |
| 2 | CARTRIDGE RETAINING NUTS | 2 |
| 3 | FLANGE STEP W/ RUBBER RING | 2 |
| 4 | HOT AND COLD CARTRIDGES | 1 ea. |
| 5 | METAL WASHER | 2 |
| 6 | LOCK NUT | 2 |
| 7 | HOT AND COLD VALVES | 1 ea. |
| 8 | SPOUT W/ SHANK | 1 |
| 9 | SPOUT RING | 1 |
| 10 | METAL WASHER | 1 |
| 11 | LOCK NUT | 1 |
| 12 | HOSE | 2 |
| 13 | TRANSFER VALVE SET | 1 |
| 14 | HAND SHOWER HOLDER ASSEMBLY | 1 |
| 15 | 45 DEGREE ELBOW | 1 |
| 16 | HAND SHOWER | 1 |
| | | |
| | | |

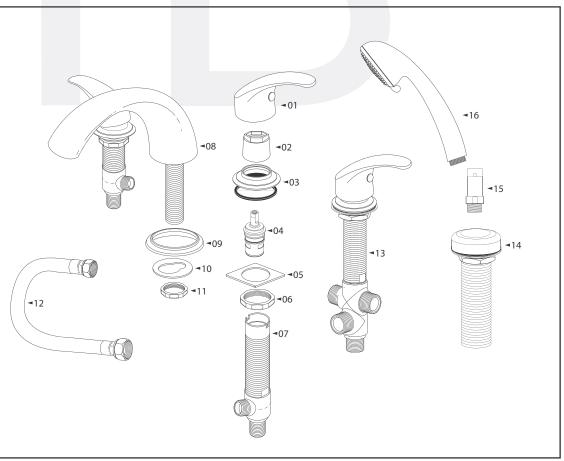

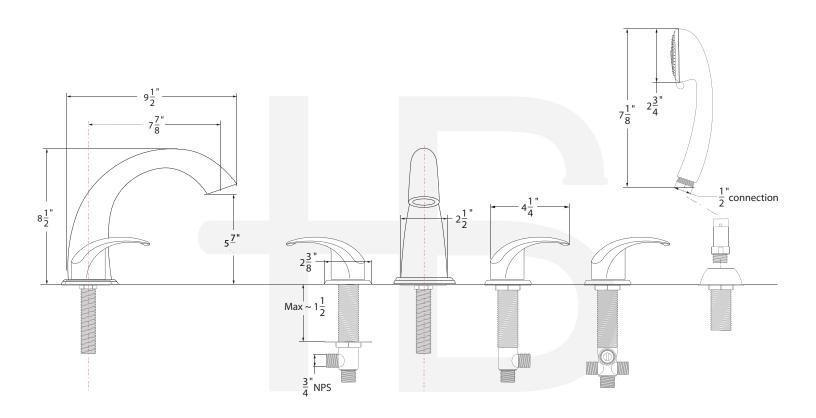

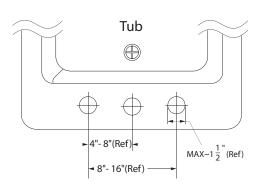

| Item Number | Туре | Date |
|-------------|-----------------------|------------|
| HB9937P | 5-pc Roman Tub Filler | 05/04/2012 |

| B | |
|-------|---|
| 9 | Model Number |
| | Warranty Information: Huntington Brass Faucets and parts are guaranteed to be free from defects in material & workmanship, leakage and drip-free during the period of normal use to the original consumer / purchaser (for residential home use) as long as the original purchaser owns their home. All "washerless" cartridges carry a 10 year limited lifetime warranty, and all "ceramic disk" cartridges carry a limited lifetime warranty. All other purchasers (including purchasers for industrial, commercial, and business related use) are warranted form a period of 5 years from the original date of purchase. A warranty period of 5 years is extended for all shower heads. The finish warranty for faucets is as follows: All plated finishes are guaranteed for as long as the original homeowner owns their home. The use of chemical cleaners, abrasive cleansers, cloths or sponges, acids, polishes or strippers, or any abnormal abuse of the product will void this warranty. |
| | The Following applies to ALL WARRANTIES: Proof of purchase (original sales receipt) from the original consumer purchaser must be made available to Huntington Brass for all warranty claims. Huntington Brass will replace, FREE OF CHARGE, during the "warranty period", any part which has proven defective in material and /or workmanship under the conditions of normal use. Replacement of parts can be obtained from your local dealer or by writing our corporate address listed below together with a copy of the original receipt. ALL TRANSPORTATION CHARGES will be PREPAID. This warranty is extensive in that it covers replacement of defective parts and finishes only. LABOR CHARGES AND/OR DAMAGES INCURRED DURING INSTALLATION, REPAIR OR REPLACEMENT CHARGES, ARE EXCLUDED FROM THIS WARRANTY. THE USE OF BATHROOM CLEANERS, TILE CLEANERS, AND /OR ANY ABRASIVE MATERIALS, UN APPROVED CHEMICALS FOR CLEANING PURPOSES WILL VOID THIS WARRANTY. Huntington Brass will not be responsible for any indirect, incidental or consequential damages, losses, injury or costs of any nature relating to the product. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusion may not apply to you. This warranty gives you specific legal rights, and you may also have other rights, which may vary from state to state. To make a warranty claim and request replacement parts, simply write to our corporate office on the address listed below. |
| | RGA- Returns Goods Authorizations: No returns will be accepted without a Return Goods Authorization (RGA) number. To receive an RGA number, you must provide the item number, quantity of items being returned, and a detailed reason for the return. The RGA number must appear on the outside of each carton being returned, for verification of receipt. Authorized returns may be subject to a minimum of 20% handling / restocking surcharge depending on the condition of the returned products, and for what reason they are being returned. ALL PRODUCTS MUST BE "DOUBLE CARTONED", PACKAGED PROPERLY, AND SHIPPED IN APPROVED SHIPPING CARTONS TO AVOID SHIPPING DAMAGES. Products returned in un-sellable condition or without original packaging will not be credited. Defective product(s) will be either repaired or replaced free of charge per our warranty conditions. No returns will be accepted for all special finish items unless defective. No returns will be accepted for discontinued items. |
| | Other Warranty Conditions: The Huntington Brass warranty does not cover product failure caused by abusive treatment, misuse, or damage due to handling or faulty installations. The warranty does not cover shipping cost, labor cost, or any other charge for such items as installation or replacement of any faucet, vanity, countertop, vitreous china component, and other component parts, or any other expense or loss. Please send your inquires, claims or warranty cards to: |
| | Attn.: Product Department 11100 Dana Circle Cypress, CA 90630 Tel : (714) 373-2788 / Fax : (714) 373-2791 |
| | |
| A B C | |
| | |
| | egister Today! |
| | ongratulations on purchasing a quality HUNTINGTON BRASS product. To validate your warranty, you must send in this registration card within 10 days of purchase. ease keep the original sales receipt with the top portion of this warranty card in the event of a warranty claim. |
| Ν | ame:Phone:(|
| | First Name Last Name |
| A | Street Address Apartment Number |
| | City State Zip |
| Ν | odel Number: Date of Purchase: |
| V | here did you buy his product ? |
| V | 'hat prompted you to buy this product? Brand image Function Style Price Finish |
| | Recommendation from sales person Other, please indicate |

Would you consider purchasing hardware / bathroom fixtures over the internet ? $\ \square$ Yes